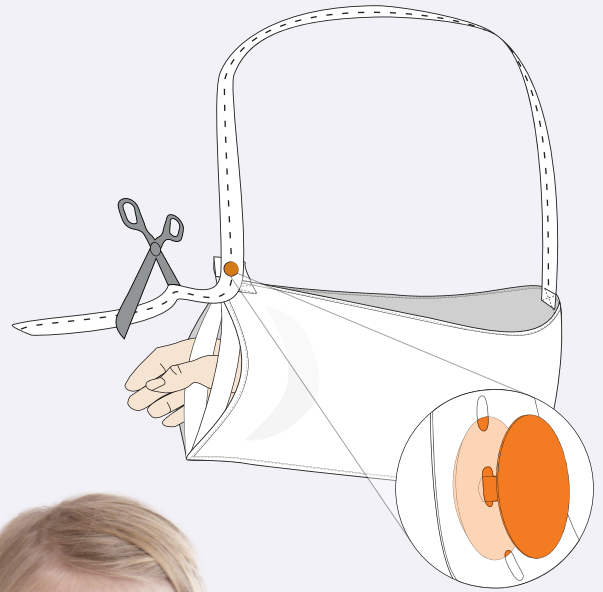
ComfortSling

ARM SLING

Art no 20111-09-50

Universal size

12 pcs.

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EN Arm sling for support of the shoulder/arm in the event of acute injuries.

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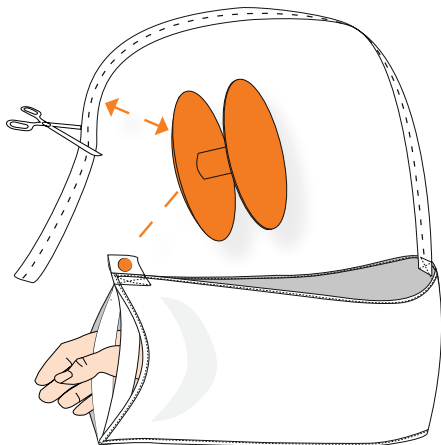
EN Place arm in the sling pocket as shown in the illustration. Pull strap back over your shoulder until buttonhole comfortable aligns with the orange button. Place button through buttonhole securely. Strap may be secured on either side of the button. If necessary the strap can be shortened to desired length.

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EN Machine wash is not recommended. If stains, wipe with a soft damp cloth or wash gently by hand in lukewarm water.

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EN In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter. Prescription, fitting and recommendations concerning use of this product must be issued by a person with medical training. For best results, follow the instructions for use. Consult a healthcare professional if used together with other medicines/aids. Serious incidents shall be reported to NordiCare and the competent authority.

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EN For support of the shoulder/arm in the event of acute injuries.

Contraindications: For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

Material composition

88% Polyester, 12% Elastane. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.