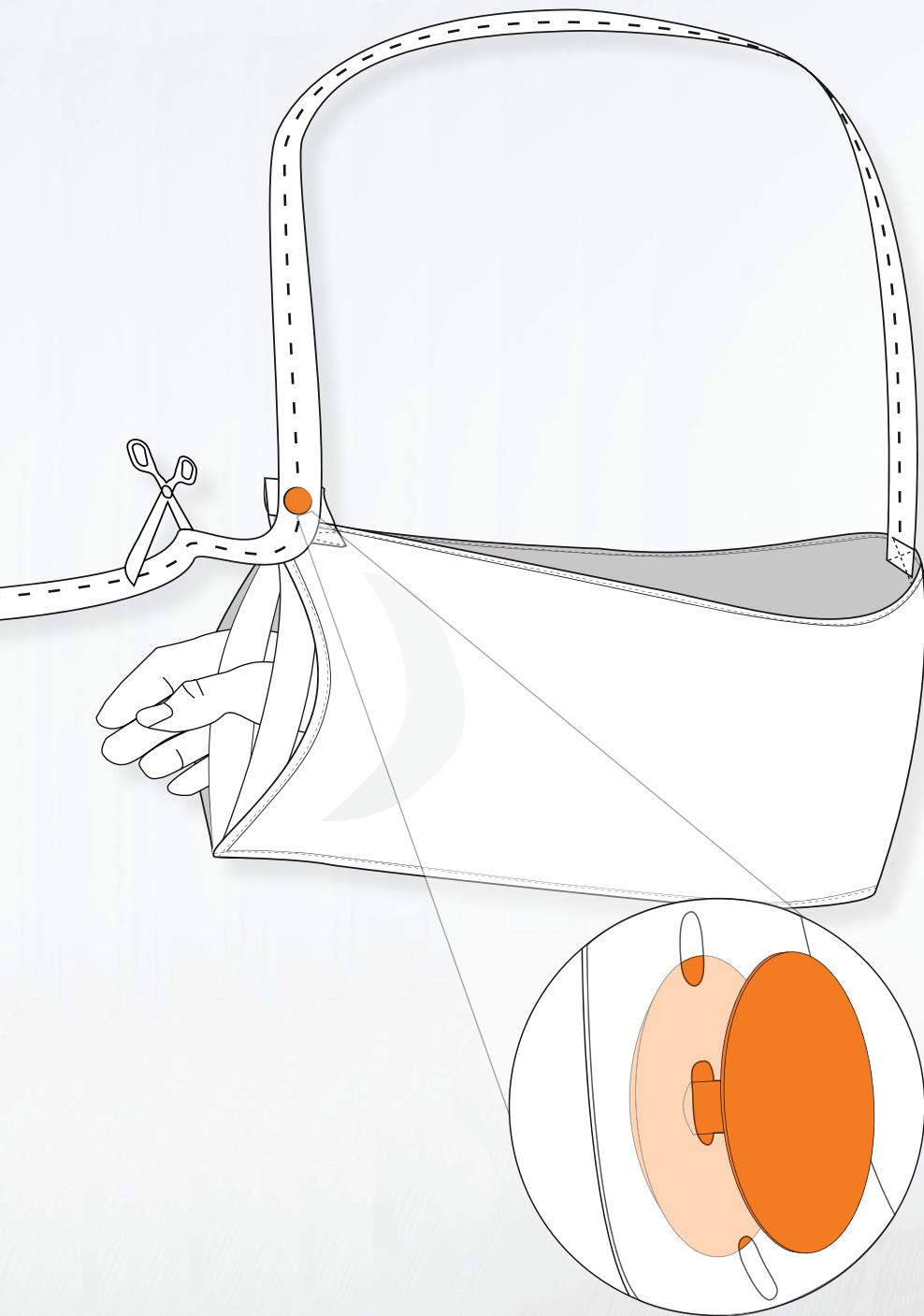
ComfortSling

Art.no 20111

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EN Arm sling for support of the shoulder/arm in the event of acute injuries.

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MD Class I

This packaging can be recycled as paper and the bag as plastic.

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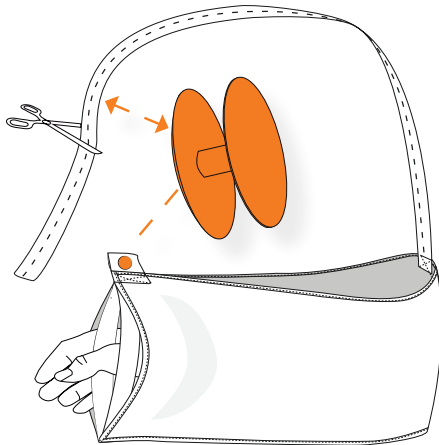
EN Place arm in the sling pocket as shown in the illustration. Pull strap back over your shoulder until buttonhole comfortable aligns with the orange button. Place button through buttonhole securely. Strap may be secured on either side of the button. If necessary the strap can be shortened to desired length.

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EN For support of the shoulder/arm in the event of acute injuries.

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EN In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter. Prescription, fitting and recommendations concerning use of this product must be issued by a person with medical training. For best results, follow the instructions for use. ComfortSling is a medical product and is intended for use by only one patient. Do not re-use for any other patients. Consult a healthcare professional if used together with other medicines/aids. Remove before taking a bath/shower. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash is not recommended. If stains, wipe with a soft damp cloth or wash gently by hand in lukewarm water.

Material composition

88 % Polyester, 12 % Elastane.

For more information please consult our product catalogue or visit www.nordicare.se.
For further questions, contact your doctor, health practitioner, medical equivalent or
NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.