Compression Garments by NordiCare

# Sandra

art no 31147

## POST SURGICAL BRA

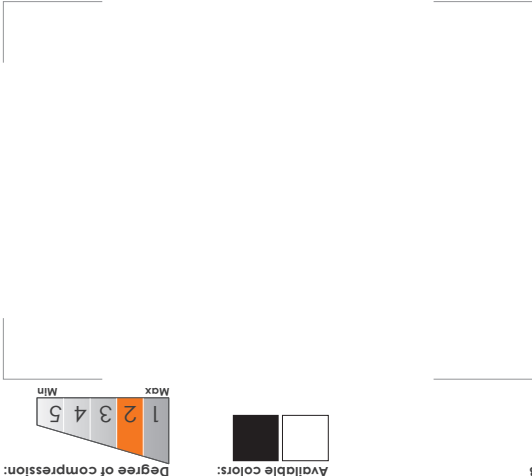
Compression level 2 - Firm

This packaging can be recycled as paper and the bag as plastic.

info@nordicare.se  
www.nordicare.se  
Phone +46 42 35 22 20  
Fax +46 42 35 22 21  
SE-263 62 Viken  
Solvägen 1,  
NordiCare Ortopedi & Rehab AB

Band	size	Cup size				
		A	B	C	D	E
cm	65	70	75	80	85	90
inch	2.6	2.8	3.0	3.2	3.4	3.6
		S (Cup A-C)	M (Cup A-C)	L (Cup A-C)	XL (Cup A-C)	2XL (Cup A-C)
		S (Cup D-F)	M (Cup D-F)	L (Cup D-F)	XL (Cup D-F)	2XL (Cup D-F)

**EN** Sandra is a knitted seamless bra with wide, adjustable straps, suitable for use after breast surgery or as a comfortable and firm wireless bra. Double material in the cup makes the bra stable and serves as a pocket for breast forms/padding.

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**EN** Stable, seamless and super soft bra. Wide, adjustable shoulder straps, back closure. Recommended for use directly after breast surgery. Also suitable as a stage-two bra or as a prosthesis bra as it has pockets for padding/breast forms.

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**EN** Machine wash at maximum 60° C (140 F), for longer durability, 40° C (104 F) is recommended. Fasten hook- and- eye closure before washing. Use of a laundry bag is recommended. Possible to tumble dry at low heat. Do not iron. If the garment has any blood stains, soak or wash in cold water. Do not use bleach, chlorine or fabric softener as this could cause the material to lose functionality.

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**EN** If you experience pinching, chafing, or color changes in the skin, remove the bra and contact the product fitter. If you recently went through breast surgery, fitting, prescribing and recommendations of use, should be done by or

under the supervision of a medical professional. Serious incidents shall be reported to NordiCare and the competent authority.

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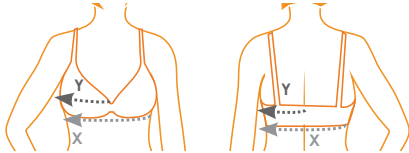
**EN** Use the size guide and size schedule to decide the right size. The fit should be tight but comfortable. The degree of compression can be adjusted by choosing a larger/smaller size.

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**EN** 1. Tight circumference measurement around bust = X. Round up or down to the nearest multiple of 5 (eg measurement 78 cm gives a bra size of 80) 2. Loose measurement over the fullest part of the breast, from the centre of the spine to the centre of the sternum = Y. Multiply this measurement by two = Y2. 3. The difference between Y2 and X decides the cup size. (Y2 - X = cup size)

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Also, see illustration above

## Material composition

Polyamide 79%, Elastane 11%, Polyester 10%. Latex free.

Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.