INSTRUCTIONS FOR Vira x3 - BINDER

art no 50227

Application

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EN Choose size and place the binder over the surged area. Begin by fastening the wide lower band **1** at the bottom of the hip. Then fasten the top band **2** and finish by fasten the middle band **3**. Adjust until it feels comfortable. The girdle should sit tight but not uncomfortably tight.

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EN Choose the size by the hip circumference, if the hip size is within an overlapping range, check the circumference of your waist:

- Narrow waistline: choose the smaller size.
- Straight body shape: choose the larger size.

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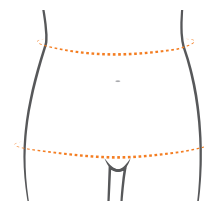
EN For postoperative care after abdominal surgery, abdominoplasty and/or liposuction. Can also be used for various types of hernia problems. Relieves wounds, provides pain relief, and enables early mobilization of the patient.

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EN The initial fitting, prescribing and recommendations for use should be done by/ under the direction of qualified medical practitioner or other professionals. Follow the instructions for best result. If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Periodically check the condition of the velcro to ensure they are free from dirt and that their fastening capabilities have not deteriorated. A qualified medical practitioner should supervise use in combination with other medical products.

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EN Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se.
For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.

NordiCare

2021-08-18

NordiCare Ortopedi & Rehab AB
Solrosvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

MD Class 1 **CE**

This packaging can be recycled as paper and the bag as plastic.