# Holger – WRIST BRACE

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**EN** Elastic wrist orthosis that provides comfortable support and relief for the CMC joints of wrist and thumb.

**NordiCare<sup>+</sup>**

## Holger

art no 40215

**WRIST BRACE**

Fits both  
Adult & Child

**NordiCare<sup>+</sup>**

Solrosvägen 1, SE-263 62 Viken, Sweden  
Phone +46 42 35 22 20, Fax +46 42 35 22 21  
info@nordicare.se, www.nordicare.se

**MD** Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

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**EN** Loosen the velcro straps. Pull the orthosis over the hand and wrist. Close the orthosis and tighten the straps. If too long, the velcro straps can be cut and adjusted.

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**EN** Instability in CMC joints. Osteoarthritis, arthritis, repetitive strain injuries. Also suitable for the treatment of carpal tunnel syndrome or for immobilization after plaster treatment.

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**EN** Machine wash at 40° C. Use a laundry bag. Dry flat. Seal the velcros. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

### Material Composition

75% Polyester, 10% Aluminum, 5% Polyurethane, 5% Elastane, 5% Nylon.

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#### **EN** User information and care:

Follow the instructions for use. If the product is prescribed by a medical professional, follow instructions as set out in the prescription throughout the rehabilitation period. Serious incidents shall be reported to NordiCare and the competent authority.

#### **Maintenance:**

Regularly check that the velcro straps are free from dirt and that their fastening capabilities have not deteriorated. Avoid using skin cream to prevent a negative impact on the bandage. Do not use the brace in the bath or shower.

#### **Always contact your doctor/fitter:**

- If you have questions.
- If you are taking medication.
- If you experience side effects such as skin irritation (remove the brace).