Tuff


art no 50200

BACK SUPPORT

LEVEL OF SUPPORT - STRONG

Size	Hip Circumference
2XL	125-135 cm
XL	115-125 cm
L	105-115 cm
M	95-105 cm
S	80-95 cm
XS	65-80 cm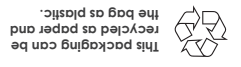

MD Class I

Phone +46 42 35 22 20,
Fax +46 42 35 22 21
Sweden
SE-263 62 Viken
Sotrovägen 1,

NordiCare®

Tuff - BACK SUPPORT

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EN 1. Loosen the elastic straps **A**.

2. Put the back support in place and fasten the Velcro at mid front over the abdomen/hip **B**, without tightening too hard.

3. Pull the elastic straps firmly outward-forward to the desired stability. Close at front **C**.

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EN Lumbal/lumbosacral back pain.

Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN To determine the size, measure hip circumference in cm. If the orthosis is to be used at waist height, use the waist measurement rather than the hip measurement.

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Material Composition

Polyester 50%, Nylon 30%, Spandex 20%. Latex free.