StomiBas

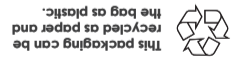
art no 50210, 50211, 50212

STOMA BINDER


2025-01-28

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature.
Dispose of the product in accordance with
the waste laws of your municipality and/or
country. The product does not present a
risk of contamination.

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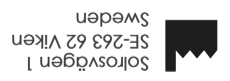EN Straight elastic stoma belt, providing a comfortable and effective support/compression around the abdomen area. The front panel is made in a non-elastic and non-fringing material and allows cutting, suitable when the stoma is to be exposed. Available in 16, 20 and 25 cm height.

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3. Pujota pussi aukon läpi (D).

EN 1. Choose the correct size and put the binder in place. Close Velcro on the side (A). Adjust until comfortable. Make sure it is not too tight.

2. StomiBas is especially suitable for making a stoma binder. The hole is to be cut in the **non-elastic material** (B). Use a template (C) and outline the size and position using a pen. N.B. The hole should not be too large, the plate/bag is to overlap the hole.

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EN Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage. Mobilization after surgery.

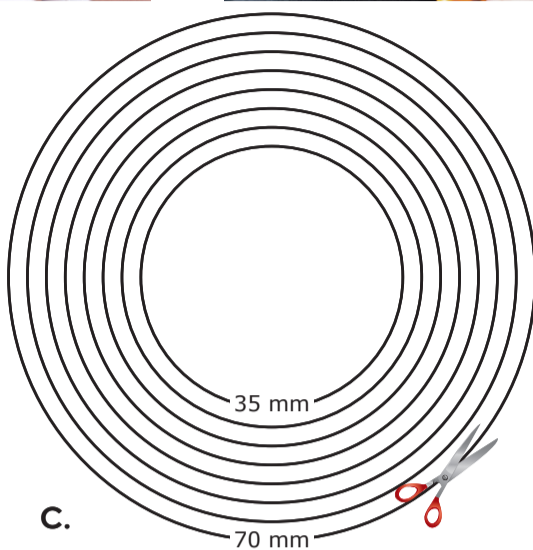
Contraindication: Incarcerated hernia.

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Material

Polyester 64%, Polyamide 23%, Elastane 10%, Polyurethane 4%.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Circumference in centimeters around the abdomen of intended placement.

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* Only for height 20 & 25 cm.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se