
StomiBas

art no 50210-00

STOMA BINDER

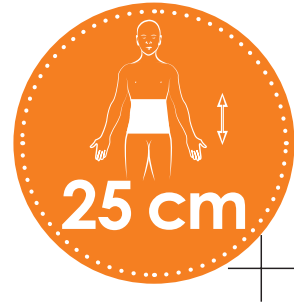2022-12-12

Available colors:

This packaging can be recycled as paper and the bag as plastic.

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EN Straight elastic stoma binder for comfortable support of stoma and peristomal hernia. The binder has a non-elastic front which is possible to cut. Easy to seal and adjust with functional handle and velcro fastening in the front. Reversible to suit stoma on both right and left side. Available in 16, 20 and 25 cm height.

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3. Pujota pussi aukon läpi **D**.

EN 1. Choose the correct size and put the binder in place. Close Velcro on the side **A**. Adjust until comfortable. Make sure it is not too tight.

2. StomiBas is especially suitable for making a stoma binder. The hole is to be cut in the **non-elastic material B**. Use the template **C** and outline the size and position using a pen. N.B. The hole should not be too large, the plate/bag is to overlap the hole.

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EN Measure circumference around intended placement.

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EN Mobilization after surgery. Stabilizing girdle for stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage. **Contraindication:** Incarcerated hernia.

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EN Machine wash 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dryclean. Do not iron. Do not tumble Dry.

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Material

Polyester 64%, Polyamide 22%, Elastane 10%, Polyurethane 4%. Latex free.

Size	Measurement in cm
XS	70-85
S	80-95
M	90-105
L	100-115
XL	110-125
2XL	120-135
3XL*	130-145
4XL*	140-155

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.