## StomiTube

art no 50214

STOMA BINDER

2023-08-23

P3285.01

STANDARD  
100  
6234-7001  
DTI  
www.oeko-tex.com

Available colors:

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**EN** 1. Put on the tube shaped binder like regular underwear. Make sure the narrower shaped part of the binder is placed in the back and that the pouch is fixated in the pockets in the lower rib.

2. Secure the pouch in the folded edge at the bottom of the binder. Wear your regular underwear underneath or on top of the StomiTube.

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**EN** To be used as a stoma band for fixation of ostomy pouches. **Contraindications:** Ostomy hernia.

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**EN** In order to determine the correct size, measure the hip circumference (at its widest point) in centimeters.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Machine wash in 60°C (140°F). The product lasts longer if washed in 40°C (104°F). Possible to tumble dry at low heat. Do not iron. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean.

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**NordiCare Ortopedi & Rehab AB**