Vidar

art no 50251

JOCKSTRAP

2022-08-31

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EN Discreet jockstrap with high comfort. Provides maximum support and protection in the scrotal area, without uncomfortable pressure of the abdomen. Seamless design, for optimum comfort.

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10093 Collelno (TO)

Italy

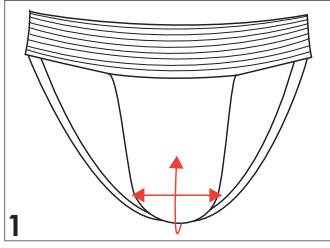
This packaging can be recycled as paper and the bag as plastic.

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EN Discreet jockstrap with maximum support. Provides lift and support as needed (1).

- To be worn directly towards the skin. Do not use the jockstrap directly on damaged/irritated skin.
- Do not use with lotions or creams.

Size	A
X Small	70-80
Small	81-90
Medium	91-102
Large	103-113

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EN Intended for treatment of edema of the scrotum, varicocele, hydrocele. Relieves and provides support for scrotum. Protects sensitive skin during activity.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (106°F). Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

Materials

Cotton 41%, Polyamide 25%, Polyester 13%, Elastane 12%, Lycra 9%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.