Alex – POST SURGICAL FACE BANDAGE

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EN Facial bandage in comfortable material with velcro closure. Easy adjustable as the velcro attaches to the entire surface of the bandage. One size fits all. 7,5 cm wide.

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EN To be used as a postoperative fixation band after facial surgery and also for compression.

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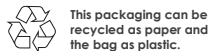
For more information please consult our product catalogue or visit www.nordicare.se.

For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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EN 1. Place the bandage around the head and tighten to required compression using the velcro closure.

2. Use the smaller straps for extra fixation, available in two different lengths.

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Materials

Polyamide 77%, Elastane 13%, Cotton 10%. Latex Free.

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EN Machine wash at 60°C (140°F). The product will last longer if washed at a lower temperature. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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Information and Care Instructions

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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