INSTRUCTIONS FOR USE

Patsy - FULL LENGTH PANTS

Part no 51201

Application

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EN 1. Make sure your skin is dry. Open the hook and eye closure.

2. Carefully pull the trousers over the narrow legs, making sure that the end of the leg ends at the desired place. Then pull the pants so that the "diamond knitting" is positioned over/around the knee.

3. Pull the pants up to the hip. Make sure that the crotch part is high/comfortable in the crotch and that the pants are centered at the back. Then pull your pants over your stomach and back. To facilitate fold the silicone edge outwards.

4. Close the hook/eyes in the side at the desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin.

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EN Recommended after plastic surgery. Compression improves circulation and reduces the risk of post-operative swelling, leading to better healing and even pain relief. Corresponding compression class 1.

Contraindications: Infection. Arterial circulatory disorders.

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Compression Garments by NordiCare

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Serious incidents shall be reported to NordiCare and the competent authority.

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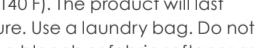
EN For best fit and correct compression it is important that measurements are taken carefully and size chosen according to the measuring table. The fit of the garment should be tight but comfortable. If the patients measurements differ from the measurement table, the garment should not be used. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required.

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EN Machine wash at 40°C (104 F). Can be washed at a higher temperature if necessary, max. 60°C (140 F). The product will last longer if washed at a lower temperature. Use a laundry bag. Do not tumble dry. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

NordiCare Ortopedi & Rehab AB
Solrosvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se