
LillaLux

art no 70513

PATELLA BRACE

Size	Circumference cm
2X-Large	53-61
X-Large	45-55
Large	38-46
Medium	33-41
Small	28-36
X-Small	25-28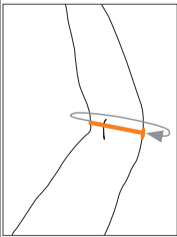

Solrosvägen 1, SE-263 62 Viken, Sweden
 Phone +46 42 35 22 20, Fax +46 42 35 22 21
 info@nordicare.se, www.nordicare.se

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EN Open wrap design for orthosis stabilization of patella. Velcro closures make the orthosis easy to adapt and adjust. C-shaped foam pillow for stabilization of patella.

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EN 1. Open the orthosis.

2. Place the C-shaped pillow on one side of the patella to prevent luxation in either the medial (inner) or lateral (outer) direction. Make sure the C-shaped pillow is placed to support along the shape of the patella.
3. Close the orthosis **3A** and tighten the straps **3B**. If too long the length of the straps can be cut and adjusted.
4. Remember to remove the C-shaped pillow during washing.

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EN Subluxation/luxation of patella and patellofemoral pain syndrome (PFSS).

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EN Machine wash at 40°C. Use a laundry bag. Dry flat. Seal the Velcros and remove the C-shaped pillow before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

Material Composition

Polyester 40%, Spandex 30%, Polyamide 15%, Nylon 10%, Styrene Butadiene Rubber (SBR) 3%, LDPE-foam 1%, Thermoplastic polyurethane (TPU) 1%.

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EN Circumference around knee, over patella.

	Size	Circumference cm	
Child	X-Small	25-28	
	Small	28-36	
	Medium	33-41	
	Large	38-46	
	X-Large	45-55	
	2X-Large	53-61	

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EN User information and care:

Follow the instructions for use. If the product is prescribed by a medical professional, follow instructions as set out in the prescription throughout the rehabilitation period. Serious incidents shall be reported to NordiCare and the competent authority.

Maintenance:

Regularly check that the Velcro straps are free from dirt and that their fastening capabilities have not deteriorated. Avoid using skin cream to prevent a negative impact on the bandage. Do not use the knee brace in the bath or shower.

Always contact your doctor/fitter:

- If you have questions.
- If you are taking medication.
- If you experience side effects such as skin irritation (remove the orthosis).

For more information please consult our product catalogue or visit www.nordicare.se.

For further questions, contact your doctor, medical practitioner or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.