## AirFoam

art no 80123

### ANKLE ORTHOSIS

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**EN** 1. Loosen the wide velcro straps around the orthosis. Place the footplate in the middle of the heel. The plate labeled "Medial" should always be on the inside of the leg. Use of a soft sock with loose elastic is recommended underneath the orthosis.

2. When necessary, adjust the Velcro straps on each side-files to match the width of the foot. Make sure the padding and tiles are just above floor level and centered opposite each other.

3. Adjust the amount of air according to the patient's needs, on both sides.

4. Tighten the lower strap around the leg first. Then tighten the top in the same way. The orthosis should sit steadily but not too hard. Place the foot in suitable laces and tie for optimum stability.

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**EN** Stadiq is a stable ankle orthosis which can be used for acute injury or prolonged instability in the ankle joint. The orthosis can also be used for prevention, for example, sports.

## Material Composition

Polyurethane 60%, Polyamide 40%. Latex free.

Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Note that the orthosis does not sit too hard to avoid stabbing.

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**EN** Hand wash, fasten the Velcro straps when washing. Clean the plastic rails with a soft damp cloth, use mild detergent.